

# **STREPSHIELD™**

## **Tongue Depressor Cough Shield**



- Simple to Use
- Fits Any Tongue Depressor
- Reversible Design
- Improved Diagnostics

# STREPSHIELD™

## Tongue Depressor Cough Shield

StrepShield™ is a unique transparent cough shield that keeps you better protected while examining or sampling the throat.

### Benefits

- Help protect your staff against airborne enemies
- Close range gag reflex protection
- Reduce transmission of respiratory illness
- Longer direct visualization
- More accurate sampling
- Improved diagnostic visibility
- Fits any tongue depressor
- Reversible design
- Single patient use

### Directions for Use

1. Bend StrepShield™ handle
2. Pinch handle onto tongue depressor
3. Use second hand for swab or light source
4. Maneuver around short edge of shield

### Ordering Information

Code Number	Description	Qty.
7446803	Tongue Depressor Cough Shield	200 / box



Please refer to the product label and / or package insert for full instructions on the safe use of these products.

StrepShield™ is a trademark of Splash Medical Devices, LLC. Patent Pending.



BSN medical Inc. 5825 Carnegie Blvd. Charlotte, NC 28209-4633  
Tel. (+1) 704 554 9933 Fax (+1) 704 358 4558 www.bsnmedical.com  
To order toll-free: BSN medical (+1) 800 552 1157

60637 RN

© 2010 BSN medical Inc.

Printed in U.S.A.

REV 08/10

